


Safety Communication

The warmer can crack or break if the unit is moved

Device/ Product Description:	Infant warmers
Brand Name:	Giraffe and Panda i-Res
Lot numbers/Serials:	<ul style="list-style-type: none"> • Giraffe Bedded Warmers (GTIN:00840682103923), • Panda iRes Warmers (GTIN:00840682103893) • (All serial numbers starting with GBW, PBW and HDJ)
Manufacturer:	GE Healthcare
Problem:	Bedside panels and latch areas on the beds may crack, break or become damaged if the unit is not properly used.
Recommendation/Actions:	<ul style="list-style-type: none"> - Inspect the warmer bedside panels (see below Figure 1A) latch areas and the areas connecting the panel to the bed (see below Figure 1B) for any cracks or damages. If any portion of the bedside panels or latches are cracked or broken (see below Figure 1C), stop use of the warmer. - GE Healthcare will replace all cracked or damaged bedside panels or latches for the Giraffe and Panda warmers and provide the additional safety labels and posters at no cost to customers. <p>You can find more information and recommendations from (HERE).</p>

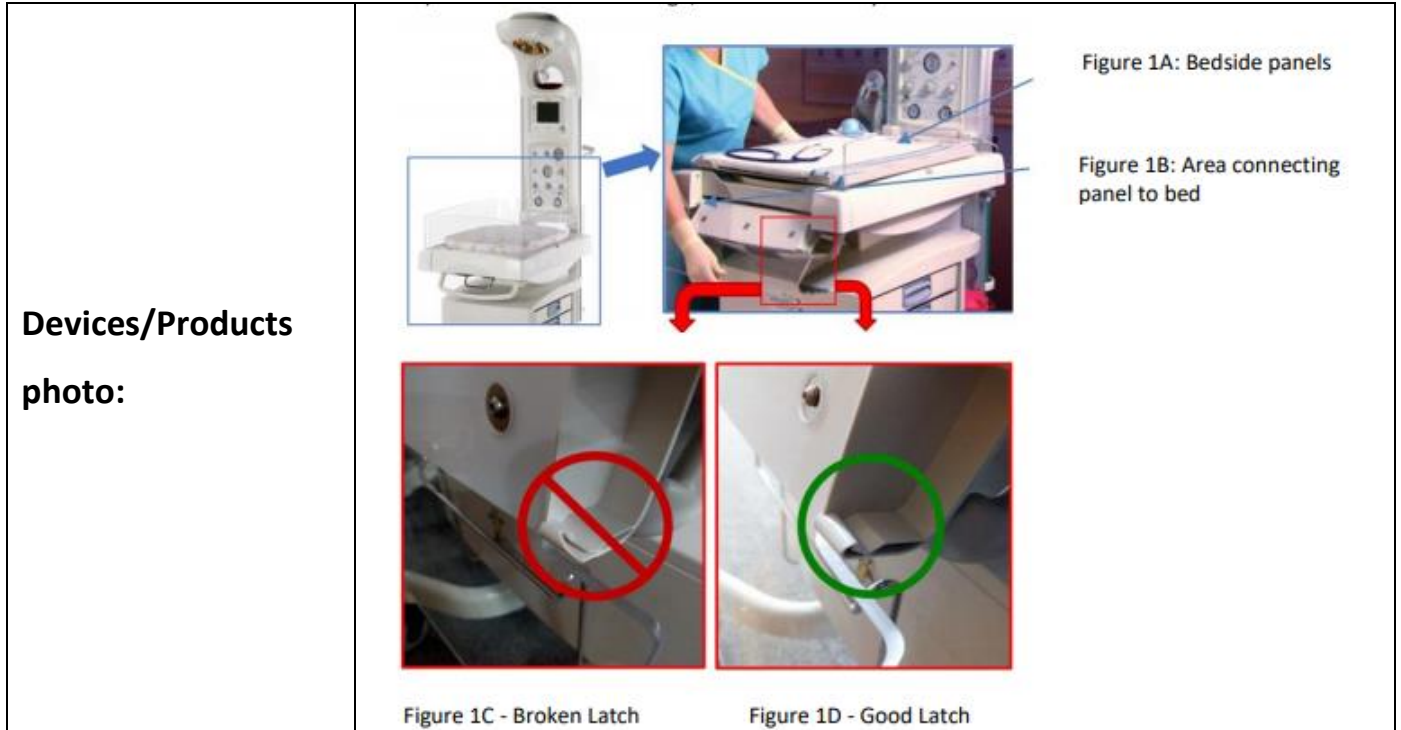

You **should** be aware of the mentioned risks in the notice and **contact** the Authorized Representative of your product for corrective action.

Healthcare Professionals should **report any adverse events** suspected to be associated with affected devices above (or other Medical Devices) to:

[National Center for Medical Devices Reporting.](#)

Medical Devices Sector

Saudi Food and Drug Authority

Postal Address: Saudi Arabia - Saudi Food and Drug Authority

4904 northern ring branch rd - Hitteen Dist.

RIYADH 13513 - 7148

Tel: +966 (11) 2038222 Ext: 2995, 2952

Fax: +966 (11) 2757245

Or

[Saudi Vigilance](#)

For latest published Recalls/Alerts, please visit ([NCMDR Website](#))

Sincerely,
NCMDR Team