

**Safety Communication**

**رسالة سلامة**

**potential for air appearing in the enteral feed pump set tubing during set-up**

<b>Device/ Product Description:</b>	Kangaroo Enteral Feeding Pump Sets																						
<b>Affected Lot Number:</b>	<p>All product manufactured before Sep 3, 2020 with lot numbers 202580052 and below</p> <table border="0"> <tr> <td>All lots before 21AXXXXXX**</td> <td>All lots before 21AXXXXXX**</td> </tr> <tr> <td>All lots before 20JXXXXXXX**</td> <td>All lots before 20HXXXXXXX**</td> </tr> <tr> <td>All lots before 21BXXXXXXX**</td> <td>All lots before 21BXXXXXXX**</td> </tr> <tr> <td>All lots before 21AXXXXXX**</td> <td>All lots before 21BXXXXXXX**</td> </tr> <tr> <td>All lots before 20LXXXXXXX**</td> <td>All lots before 21BXXXXXXX**</td> </tr> <tr> <td>All lots before 21BXXXXXXX**</td> <td>All lots before 21BXXXXXXX**</td> </tr> <tr> <td>All lots before 20KXXXXXXX**</td> <td>All lots before 21AXXXXXX**</td> </tr> <tr> <td>All lots before 21BXXXXXXX**</td> <td>All lots before 20LXXXXXXX**</td> </tr> <tr> <td>All lots before 21BXXXXXXX**</td> <td>All lots before 21BXXXXXXX**</td> </tr> <tr> <td>All lots before 20FXXXXXXX**</td> <td>All lots before 20AXXXXXX**</td> </tr> <tr> <td>All lots before 21AXXXXXX**</td> <td>All lots before 21BXXXXXXX**</td> </tr> </table>	All lots before 21AXXXXXX**	All lots before 21AXXXXXX**	All lots before 20JXXXXXXX**	All lots before 20HXXXXXXX**	All lots before 21BXXXXXXX**	All lots before 21BXXXXXXX**	All lots before 21AXXXXXX**	All lots before 21BXXXXXXX**	All lots before 20LXXXXXXX**	All lots before 21BXXXXXXX**	All lots before 21BXXXXXXX**	All lots before 21BXXXXXXX**	All lots before 20KXXXXXXX**	All lots before 21AXXXXXX**	All lots before 21BXXXXXXX**	All lots before 20LXXXXXXX**	All lots before 21BXXXXXXX**	All lots before 21BXXXXXXX**	All lots before 20FXXXXXXX**	All lots before 20AXXXXXX**	All lots before 21AXXXXXX**	All lots before 21BXXXXXXX**
All lots before 21AXXXXXX**	All lots before 21AXXXXXX**																						
All lots before 20JXXXXXXX**	All lots before 20HXXXXXXX**																						
All lots before 21BXXXXXXX**	All lots before 21BXXXXXXX**																						
All lots before 21AXXXXXX**	All lots before 21BXXXXXXX**																						
All lots before 20LXXXXXXX**	All lots before 21BXXXXXXX**																						
All lots before 21BXXXXXXX**	All lots before 21BXXXXXXX**																						
All lots before 20KXXXXXXX**	All lots before 21AXXXXXX**																						
All lots before 21BXXXXXXX**	All lots before 20LXXXXXXX**																						
All lots before 21BXXXXXXX**	All lots before 21BXXXXXXX**																						
All lots before 20FXXXXXXX**	All lots before 20AXXXXXX**																						
All lots before 21AXXXXXX**	All lots before 21BXXXXXXX**																						
<b>Manufacturer:</b>	Covidien, Cardinal Health LLC																						
<b>Problem:</b>	<p>The issue relates to air entry into the feed set tubing, which appears as air space in the tubing.</p> <p>Air entry into the feed set tubing may potentially result in the following health consequences: vomiting, dehydration, hypoglycemia, abdominal pain, and abdominal distention in patients.</p>																						
<b>Recommendation /Actions:</b>	<ul style="list-style-type: none"> <li>• Make sure that this document is reached to the end-users.</li> <li>• Follow the following steps to check the feed set: <ul style="list-style-type: none"> <li>- Press “Run” on the pump after installing and priming the feeding set.</li> <li>- After 30 seconds, watch the tubing between the pump and the patient to check if the air getting into the line</li> <li>- It is not a problem, to see a small amount of air in the feed tubing (approximately 1.3 cm [1/2 inch])</li> </ul> </li> </ul>																						

- **Do not use the feeding set if you see the amount of air measuring greater than 2.5 cm (1 inch). Please replace the set.**
- **Do not use the feeding set if you see several air bubbles its total measure greater than 2.5 cm (1 inch). Please replace the set.**



- Contact the authorized representative for any needed support.

For more information, Please click [here](#).

If you think you had a problem with your device or a device your patient uses, please report the problem to SFDA through:

[NCMDR](#)  
[Vigilance system](#)  
 (19999) unified call center

**Devices/Products photo:**



**Authorized Representative Details**

AR name:	International Medical Supplies Co.
Assigned Contact Person:	Ammar Abdullah Kaabi
Mobile/Phone:	011- 243 9384
Email:	<a href="mailto:akaabi@tibbiyah.com">akaabi@tibbiyah.com</a>