

**Safety Communication**

**رسالة سلامة**

**Potential for Medication Overdose with ENFit Low Dose Tip Syringe**

<p><b>Device/ Product Description:</b></p>	<p>ENFit Low Dose Tip Syringe</p>
<p><b>Problem:</b></p>	<p>The potential for overdose if the user does not clear the moat area around the tip of the ENFit LDT syringe before administering a medication.</p>
<p><b>Recommendation /Actions:</b></p>	<p><b>Recommendations for Health Care Providers and Patients to optimize the accuracy of using the ENFit Low Dose Tip Syringe:</b></p> <ul style="list-style-type: none"> <li>• Ensure the syringe is free of air bubbles and the moat is free of fluids by flicking the tip of the syringe.</li> <li>• Use a filling adapter (e.g ENFit compatible cap) to prevent fluid from entering the moat area.</li> <li>• Be aware that using a medicine cup to fill may cause the fluid or medications to enter moat of the syringe and lead to possible overdose.</li> <li>• Use a new syringe to flush the medication after administrating to avoid overdose due to the dead space in the syringe.</li> </ul> <p>For more information, please click <a href="#">here</a>.</p> <p>If you think you had a problem with your device or a device your patient uses, please report the problem to SFDA through:  <a href="#">NCMDR</a>  <a href="#">Vigilance system</a>  (19999 )unified call center</p>

