Date: 15 Apr 2024

Reference Number: SG-2404-418-H

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Medical Devices Sector
National Center for Medical Devices Reporting

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Title	Recommendations for Healthcare Providers regarding the safe and effective use of Autoclave Devices	العنوان
Medical Device Description	Autoclave Devices	اسم ووصف الجهاز/المستلزم الطبي
Manufacturer	All	اسم المصنع
Authorized Representative	All	الممثل المعتمد
Medical Devices Marketing Authorization (MDMA)	All	إذن التسويق
Potential /Associated risks	<p>According to the SFDA's survey report on autoclave devices, which was distributed to Saudi healthcare providers to seek feedback on their experience with the autoclave devices.</p> <p>The report identified a number of possible risks and concerns. As a result, the SFDA recognized the importance of emphasizing safety recommendations to individuals who use steam sterilization devices to avoid possible risks related with autoclave usage, such as heat burns and scalds.</p>	المخاطر المحتملة/ المرتبطة بالجهاز أو المستلزم الطبي
Recommendations	<p>Healthcare providers must:</p> <ol style="list-style-type: none"> 1. Use autoclave devices that approved by SFDA with an MDMA certificate. 2. Specialized, trained, and qualified to use autoclave devices by receive comprehensive training on how to operate the autoclave correctly. 3. Follow the manufacturer's recommendations for proper installation, operation, cleaning, and maintenance. 4. Place the user manual near the autoclave device being used. 5. Wear personal protective equipment, such as gloves, goggles, and lab coats, when using the device. They acts as a barrier, shielding from potential hazards. 6. Ensure the autoclave is well-ventilated to prevent moisture and heat buildup. 7. Regularly monitor the autoclave's pressure and temperature gauges during operation. 	التوصيات
For Reporting	  	للإبلاغ